



FAQ: What do you recommend for keeping the pelvic sensor secure on sweaty horses?

A: If the horse has long enough hair, then the pelvic clip can often be used alone on sweaty or wet horses. But if the hair is not long enough, or you find that you are not getting good grip with the clips, then use the pelvic sensor pads with some additional preparation.

It is always best to start with a clean, dry horse.

1. Before the horse sweats, spray a **human or veterinary medical grade adhesive** on the area of the tuber sacrale where the pelvic sensor pad will be placed. Let it dry.
*If the horse is already wet, towel dry and then apply the spray adhesive.
2. Place the pelvic sensor pad over the tuber sacrale as usual.
3. Spray the top of the pelvic sensor pad with adhesive and let dry. This will allow the pelvic clips to grab and hold fast.
4. Apply the pelvic clip.



*Another method is to use **kinesiology tape (KT)**. This may be more suitable when the horse sweats excessively, or for longer evaluations.*

1. **Before** the horse sweats, spray the medical grade adhesive on the area of the tuber sacrale where the pelvic sensor pad will be placed. Let it dry.
2. Apply KT over the tuber sacrale. Rub down over the tape to warm it and activate the adhesive.
3. Spray adhesive over K Tape, let dry.
4. Apply the pelvic sensor pad.
5. Spray adhesive over the pad, let dry.
6. Apply the pelvic clip.
7. Follow individual KT instructions for proper removal.

*To remove any residue of KT or pelvic sensor pads, use a **medical adhesive remover** and alcohol.*

There are many good human and veterinary medical grade adhesives available on the market. We are currently testing various brands in field settings. We'll let you know what the winner is in the coming weeks!